



Fixture Level Impression - Opened Tray Technique

Step 1

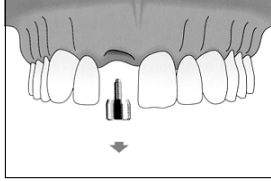


Remove the healing cap.

Step 2

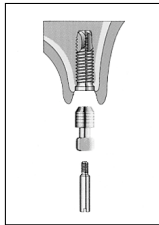

Transfer copings for the fixture level are positioned using a guide pin for connection.

Step 3

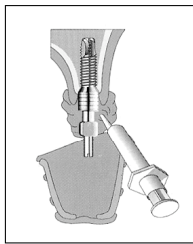

An elastomeric impression material is injected around the impression copings prior to positioning the tray. After set, remove the impression.

Step 4

Unscrew transfer coping by hand.

Images courtesy of Branemark.



Bay View Dental Laboratory

1207 Volvo Parkway
Chesapeake, VA 23320-7654

757.583.1787

info@BVDL.com

www.BVDL.com